


## User Manual

### Removing the filter holder

**Keep in mind:** close the rear tap! Lift the spout dispenser and remove the filter holder section.


**1**

Close the tap at the rear by DSPENS.

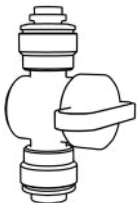


**2**

Raise the spout.

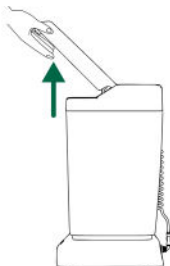


**3**

Push the black lever forward.

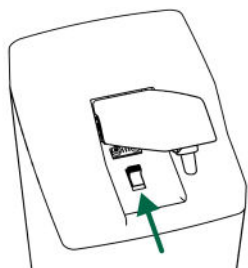

**4**

Lift the top of the DSPENSER.

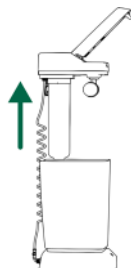


### Changing filter

Turn the filter clockwise to install and anti clockwaise to dismount


**5**

Rotate the filter counter-clockwise until it is unlocked.

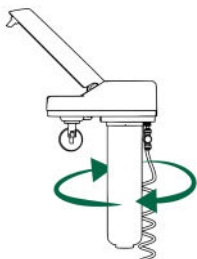


**6**

Remove the filter

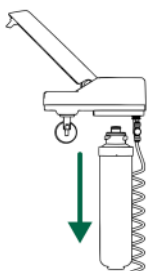


**7**

Place the new filter in its seat.

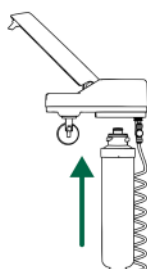


**8**

Rotate the filter counter-clockwise until it is locked.

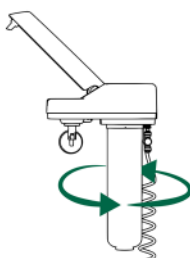


**9**

Pull the lever towards you to lock the system

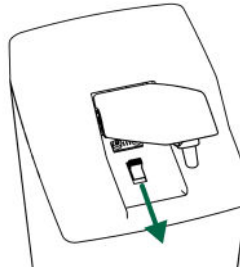

**10**

Open the tap.

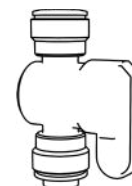


### INPUT and OUTPUT

**WATER SUPPLY:** Place the pipe coming from the water supply or osmosis into the tap.

**OVERFLOW PIPE:** To prevent flooding in case of failure, extend the silicone hose behind the filter and connect it to the sink drain

**WATER OUTPUT:** connect the sterilizer or an accessory such as the Water Gun

**POWER SUPPLY:** connect the power supply to the input connector.

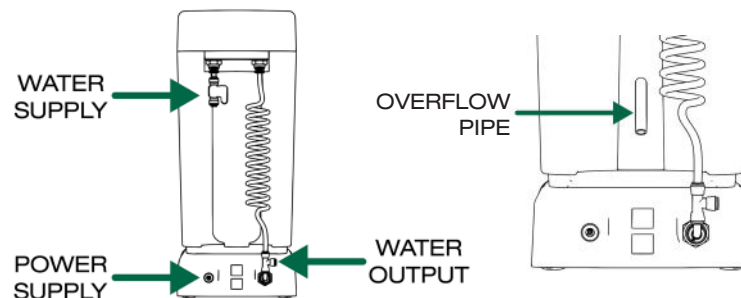

### Pump activation

Press the button on the base.




### Dispensing via spout

Press the lever down to dispense.



### Output connection

To connect the sterilizer or an accessory such as the Water Gun, insert the 1/8" tube into the appropriate fitting as in the image and position the blue clip to block the tube

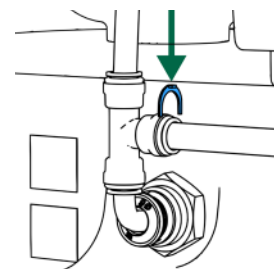

### Cleaning - Maintenance

Clean with a cloth approximately once a week or any time filter is changed.